


QUICK REFERENCE GUIDE - ALLOGRAFT SKIN RECOVERY TECHNIQUE

amalgatome

MD




1) Set the depth adjustment knob to the desired excision depth.

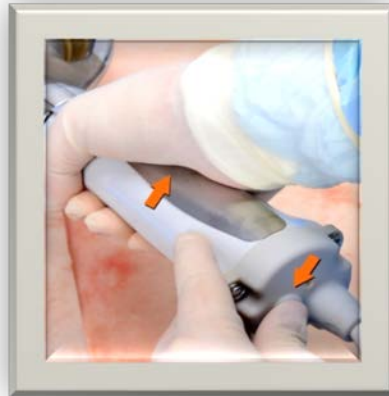


2) Engage activation LEVER on handle while pressing the activation BUTTON. If activation lever is held down for 5 seconds without pressing the activation button, device will deactivate and activation lever must be released to reset and restart the device.



3) Place the head assembly at desired skin graft starting point. Apply slight downward pressure to ensure cutting edge of blade maintains contact with the skin graft site.

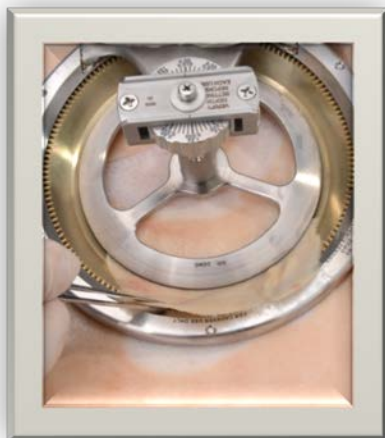

4) With the blade spinning, pull device in desired direction for skin graft. Maintain comfortable speed to ensure desired thickness and width is recovered.

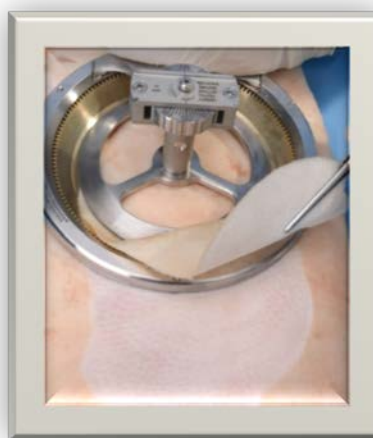

5) As skin graft emerges from blade mechanism, use forceps to gently lift skin. Do not stretch or pull skin.




6) When skin graft recovery is complete, set the depth adjustment knob to the closed position for safe blade removal.

QUICK REFERENCE GUIDE - OPERATION AND ASSEMBLY


amalgatome | MD



1) Set depth gauge to closed position prior to the blade installation.

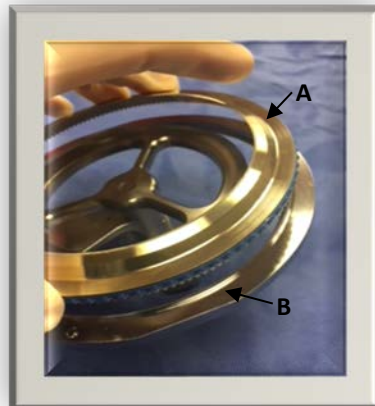

2) Turn the Amalgatome MD Lever side down. Place blade (A) into housing/frame (B) and slightly turn blade in any direction for smooth rotation.

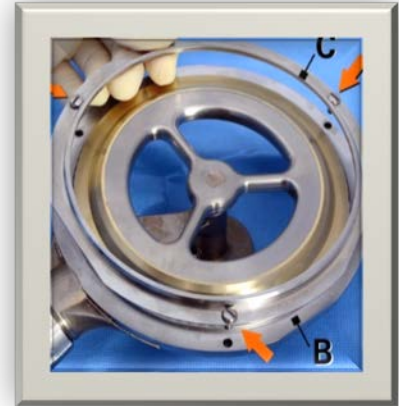

3) Place blade locking ring (C) into housing/ frame (B) and ensure the three (3) screws align with screw holes. Tighten with screwdriver included with product.



4) Attach power cord to back of power supply and plug into electrical outlet.



5) Remove protective cap from end of handpiece power cord and plug into power supply by aligning red dots.



6) Turn power switch to ON. The product is ready to operate.

Contact Us:
10804 Green Rd.
Wakeman, OH. 44889
Phone: 800-243-6049

Sales: sales@exsurco.com
Information: info@exsurco.com
Service/ Support: support@exsurco.com
www.exsurco.com

FOR CADAVERIC USE ONLY

Exsurco Medical is a Medical Device Company registered with the FDA and ISO 13485 Certified.

Amalgatome and Exsurco are registered trademarks of Exsurco Medical Inc. © 2018 Exsurco Medical, Inc. All rights reserved.