




## Operation & Assembly

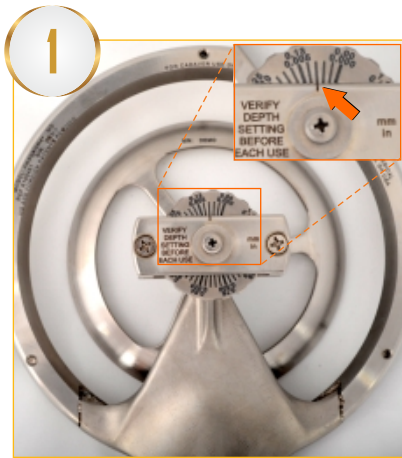

1 Set depth gauge to .003" (closed position) prior to the blade installation.

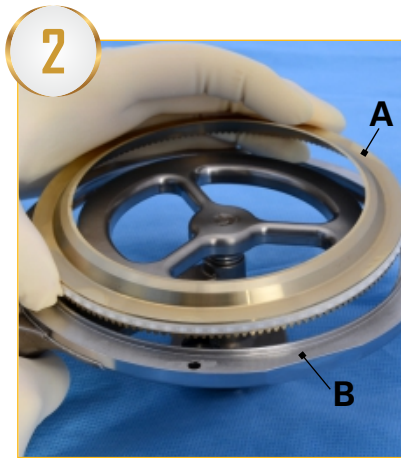

2 Turn the Amalgatome® MD lever side down. Place blade (A) into housing/frame (B) and slightly turn blade in any direction for smooth rotation.

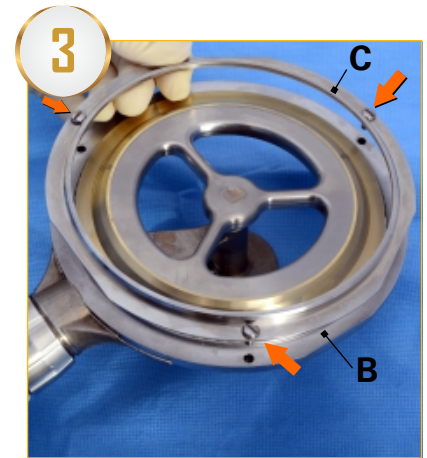

3 Place blade locking ring (C) onto housing/frame (B) and ensure the three (3) screws align with screw holes. Tighten with screwdriver included with device.



4 Attach power cord to back of power supply and plug into electrical outlet.



5 Remove protective cap from end of handpiece power cord and plug into power supply by aligning red dots.



6 Turn power switch to ON. The device is ready to operate.

### Contact Us:

For additional information:  
support@exsurco.com | 800-243-6049

## Skin Recovery Technique



1 Set the depth adjustment knob to the desired excision depth.



2 Engage activation LEVER on handle while pressing the activation BUTTON. If activation lever is held down for 5 seconds without pressing the activation button, device will deactivate and activation lever must be released to reset the device.




3 Place head assembly at desired skin graft starting point. Apply slight downward pressure to ensure cutting edge of the blade maintains contact with the skin graft site.



4 With the blade spinning, pull device in desired direction for skin graft. Maintain comfortable speed to ensure desired thickness and width is recovered.



5 As skin graft emerges from blade mechanism, use forceps to gently lift skin. Do not stretch or pull skin.

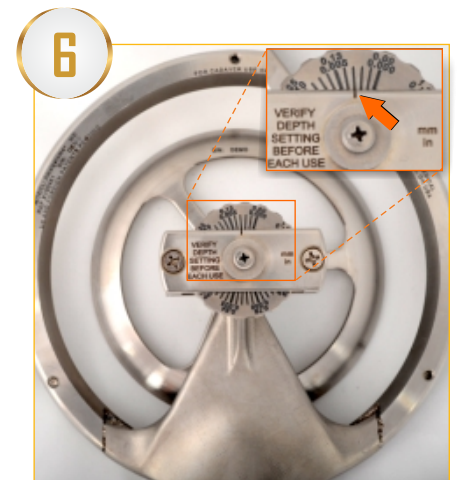

6 When skin graft recovery is complete, set the depth adjustment knob to the closed position (.003") for safe blade removal.

For additional information please refer to the Operation & Instruction Manual

**EXSURCO**<sup>®</sup>  
medical