


amalgatome | MD


Sterilization Options



Sterilization Tray

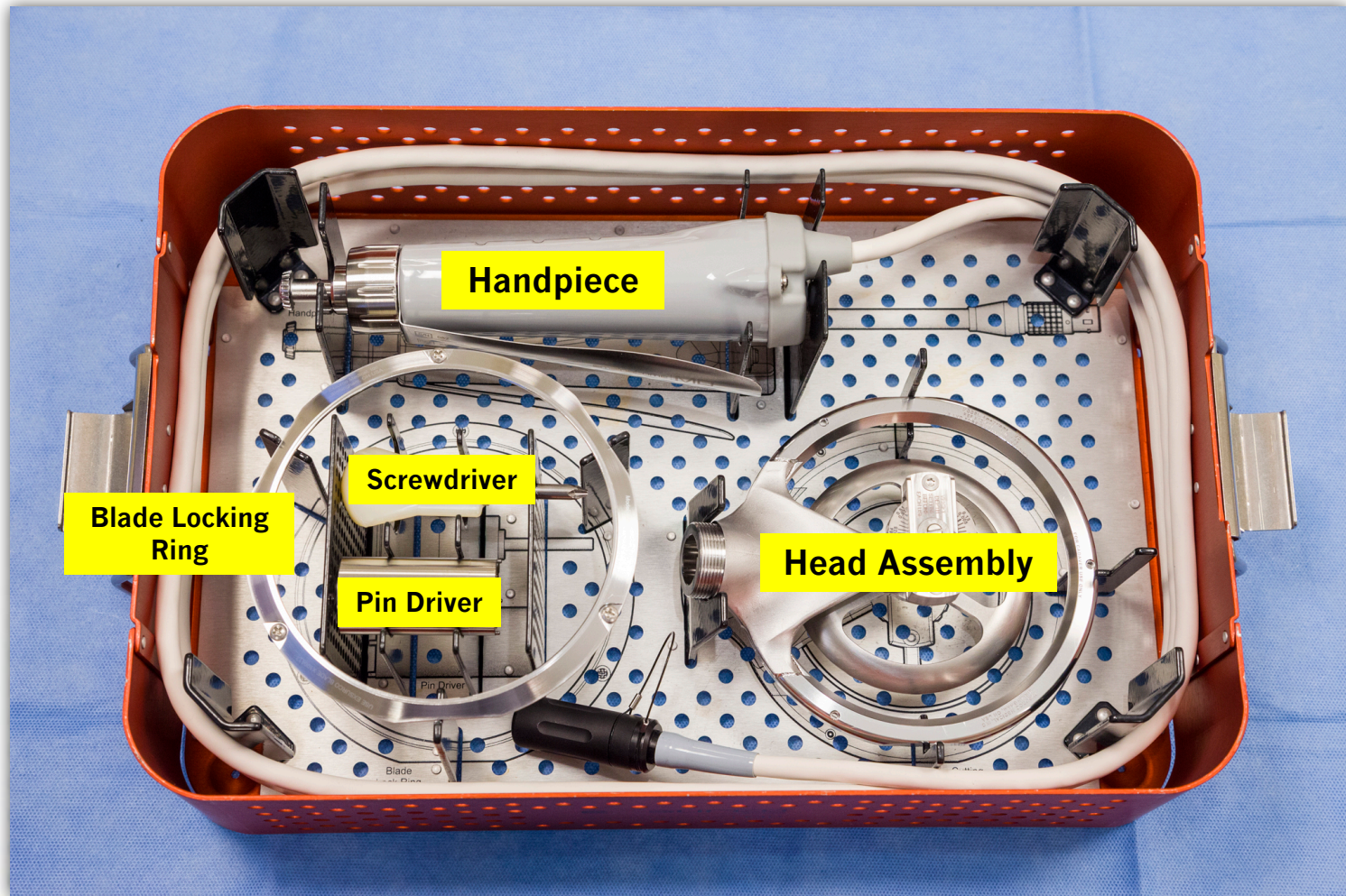


Sterilization Tray

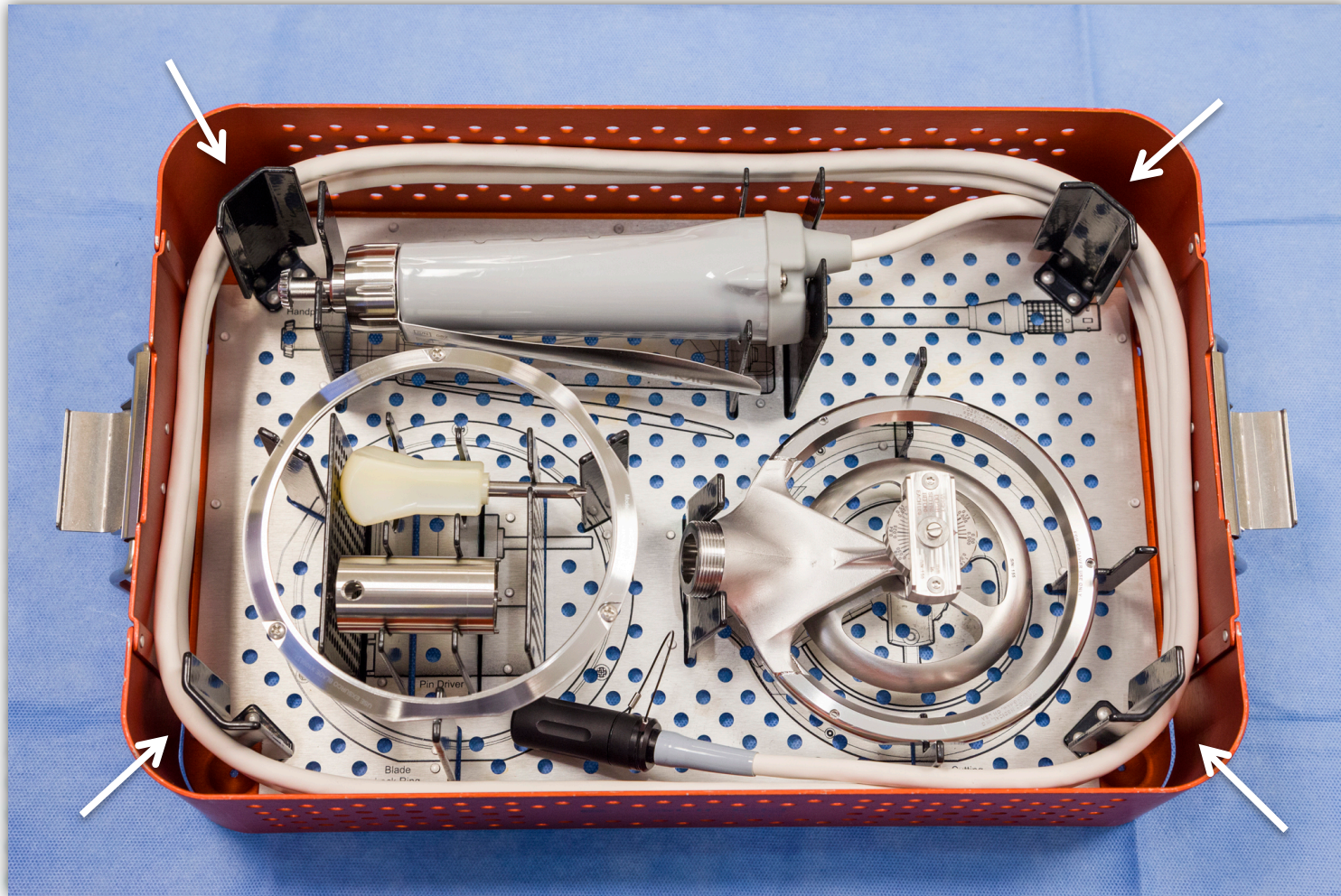

Note: as the power cord is wrapped around the outside of the tray, make sure not to apply too much tension at the corners of the tray.

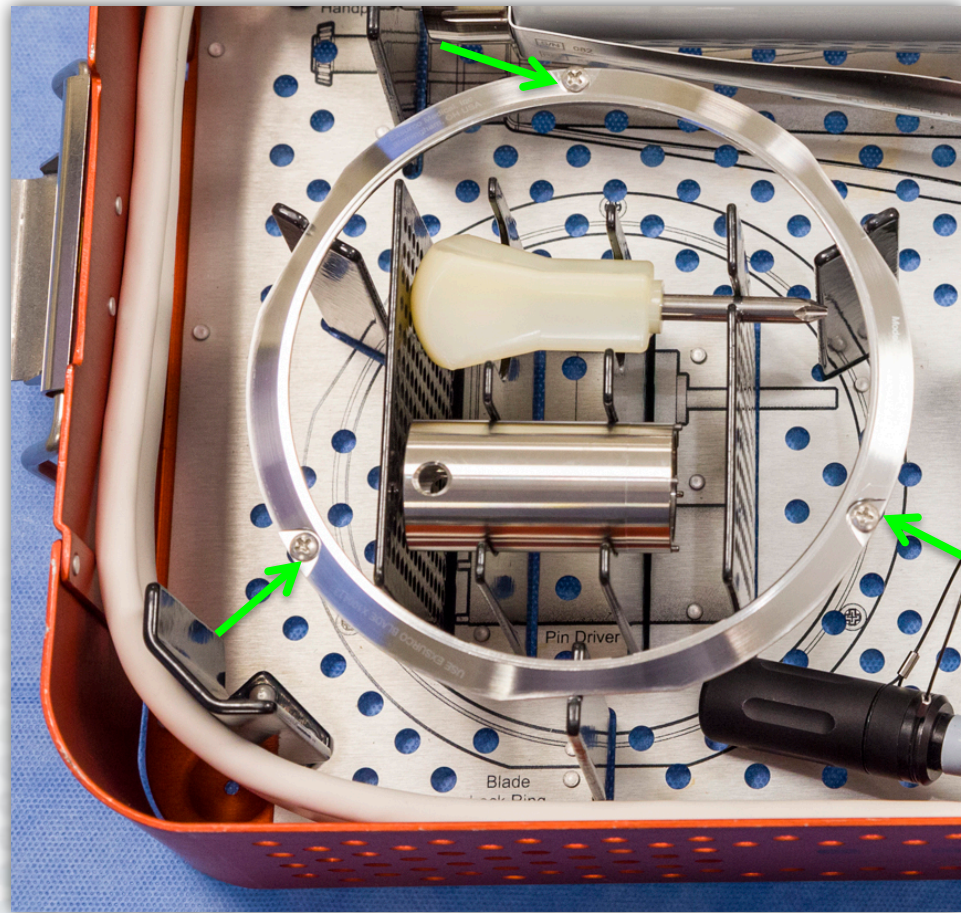


Sterilization Tray



Ensure the blade locking ring is placed inside the tray with the screw heads facing upward.



Sterilization

- Place steam sterilization indicator strip inside tray 
- Wrap the sterilization tray in CSR wrap 
- Size 45" x 45" or larger works best
- Secure closed with steam indicator tape 
- Label as appropriate



Sterilization Options



Cycle Type	Gravity, <i>Hospital</i>	Cycle Type	Prevacuum, <i>Hospital</i>
Set Point Temperature	135°C	Set Point Temperature	132°C
Full Cycle Dwell Time	10 minutes	Full Cycle Dwell Time	4 minutes
Dry Time	30 minutes	Dry Time	20 minutes

- The Guidance for industry and FDA staff, reprocessing medical devices in a healthcare setting: March 17, 2015
- ANSI/AAMIST8 Hospital Steam Sterilizers
- ANSI/AAMIST79 Comprehensive Guide to Steam Sterilization and Sterility assurance in healthcare facilities
- AAMITRI 12 Designing, testing and labeling reusable medical devices for reprocessing the health care facilities
- ANSI/AAMI/

Note: the device is to be steam processed only one time per use.

