

amalgatome | MD

Quick Reference Guide



Quick Reference Guide




Quick Reference Guide


QUICK REFERENCE GUIDE

amalgatome | MD


Operation & Assembly

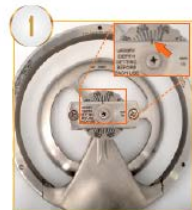

Set depth gauge to .003" (closed position) prior to the blade installation.

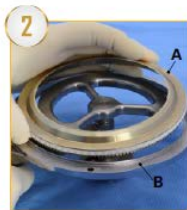

Turn the Amalgatome® MD lever side down. Place blade (A) into housing/frame (B) and slightly turn blade in any direction for smooth rotation.

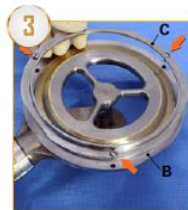

Place blade locking ring (C) onto housing/frame (B) and ensure the three (3) screws align with screw holes. Tighten with screwdriver included with device.



Attach power cord to back of power supply and plug into electrical outlet.



Remove protective cap from end of handpiece power cord and plug into power supply by aligning red dots.



Turn power switch to ON. The device is ready to operate.

Contact Us:

For additional information:
support@exsurco.com | 800-243-6049

Quick Reference Card
Part #X1005/4

EXSURCO
medical

QUICK REFERENCE GUIDE

amalgatome | MD

Skin Recovery Technique



Set the depth adjustment knob to the desired excision depth.



Engage activation LEVER on handle while pressing the activation BUTTON. If activation lever is held down for 5 seconds without pressing the activation button, device will deactivate and activation lever must be released to reset the device.



Place head assembly at desired skin graft starting point. Apply slight downward pressure to ensure cutting edge of the blade maintains contact with the skin graft site.



With the blade spinning, pull device in desired direction for skin graft. Maintain comfortable speed to ensure desired thickness and width is recovered.



As skin graft emerges from blade mechanism, use forceps to gently lift skin. Do not stretch or pull skin.



When skin graft recovery is complete, set the depth adjustment knob to the closed position (.003") for safe blade removal.

For additional information please refer to the Operation & Instruction Manual

Amalgatome and Exsurco are registered trademarks of Exsurco Medical, Inc.
© 2013 Exsurco Medical. All rights reserved.

EXSURCO
medical



Operation & Instruction Manual

- Included with each Amalgatome MD
- Also available via:
 - Phone: (800) 243-6049
 - Email: support@exsurco.com
 - Online: www.exsurco.com/amalgatome-md

